

Solfy Series

Tip Guide

External Thread Type: SC-3000 and SC-7000

Internal Thread Type: SC-7000T2



Table of Contents


Tip Type	3
Thread Shape	3
Power Range	3
Maximum Power Level	3
Tip Lineup	4
External Thread Type	7
Scaling	8
Perio	12
Endo	20
Retro Endo	29
Others	33
Internal Thread Type	35
Scaling	36
Perio	39
Endo	45
Retro Endo	52
Others	54
Consumable and Replacement Parts	55
Tip Power Guide	56

Tip Type

Thread Shape

The thread shapes of the scaler tip differ between SC-3000, SC-7000, and SC-7000T2. Scaler tips for SC-3000 and SC-7000 have external threads, and scaler tips for SC-7000T2 have internal threads; these are not compatible with each other.

There is a laser-marked “Type T” on SC-7000T2 handpieces. Be sure to check the thread type and use the correct scaler tips for your handpiece.

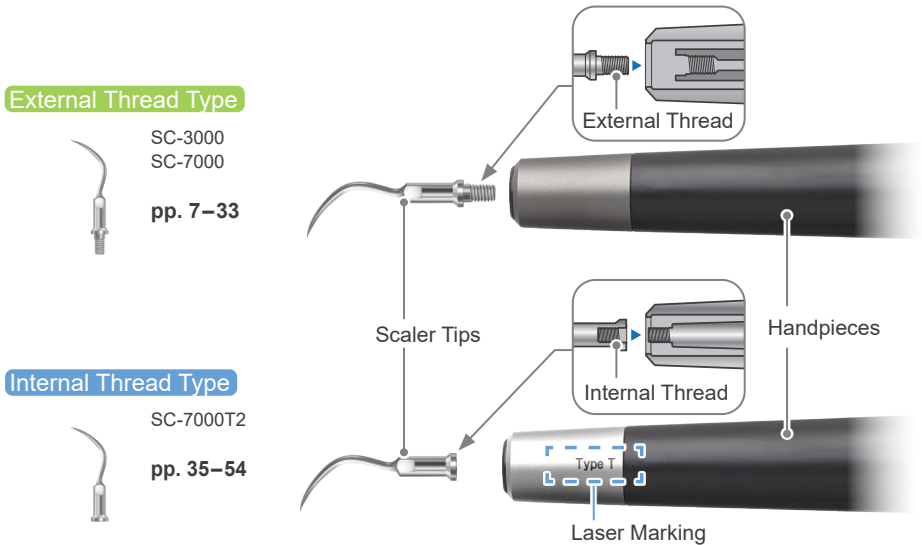

Power Range

SC-3000 : 0 – 7
SC-7000 & SC-7000T2 : 0 – 25

Maximum Power Level


Do not use scaler tips in excess of their maximum (est.) power level. Using the scaler at powers higher than those recommended or applying the scaler tip with excessive force could result in overheating the treatment area or scaler tip, and damaging the scaler tip.

Tip Lineup

Scaling

External Thread Type pp. 8–11



Internal Thread Type pp. 36–38

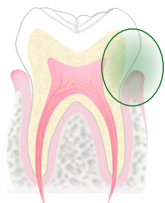

Perio (periodontal)

External Thread Type pp. 12–19



Internal Thread Type pp. 39–44

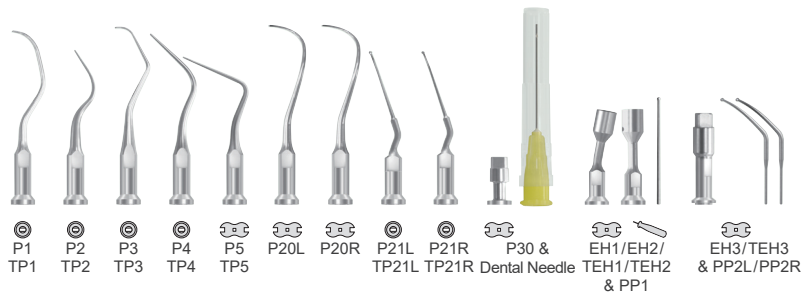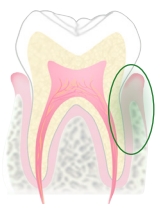





Endo (endodontic)

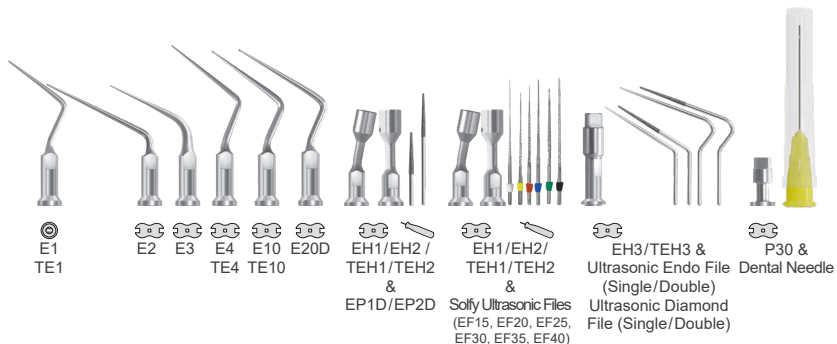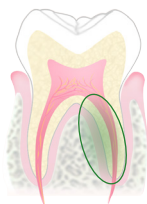External Thread Type pp. 20–28

Internal Thread Type pp. 45–51






Torque Wrench Symbol:  This designates tips that are attached using the torque wrench .

Tip Tool Symbol:  This designates tips that cannot be attached using the torque wrench. Use the tip tool  instead.

File Tool Symbol:  This designates files and points that can be installed using the file tool .

Retro Endodontic Tips (retrograde endodontics) External Thread Type pp. 29–32 Internal Thread Type pp. 52–53

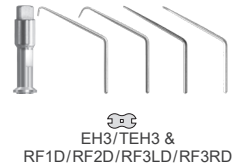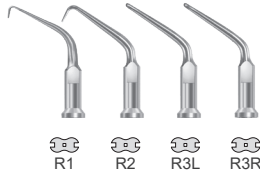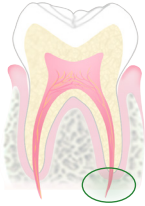

Others

External Thread Type p. 33
Internal Thread Type p. 54




Accompanying Tips

SC-7000


SC-7000T2



Laser Marking

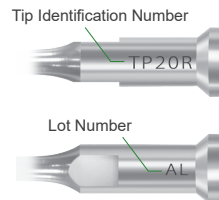

The tip identification number and lot number are laser marked on each tip.

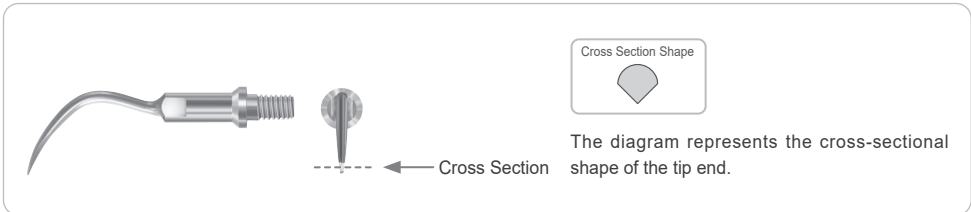

External Thread Type

SC-3000

SC-7000





Scaling

Scaling Tips

S1

Universal Tip S1

- Scaling, irrigation, and cleaning

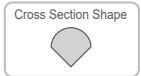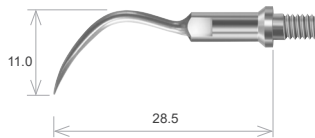

Maximum Power (est.)



SC-3000	6
SC-7000	20

Code No.: 8710503

S2

Straight Tip S2

- Calculus removal (mainly inside periodontal pocket)
- Irrigation (cavities, root canals, blind pockets, periodontal pockets)
- Debris removal (cement, gutta-percha, sealer, fragments, etc.) from canals

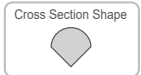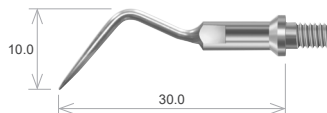

Maximum Power (est.)

SC-3000	6
SC-7000	20

Code No.: 8713707


Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.



S3

Scaling Tip S3

- Scaling, irrigation, and cleaning (mainly fissures)

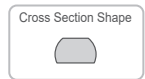


Maximum Power (est.)	
SC-3000	6
SC-7000	20

Code No.: 9186247

S4

Scaling Tip S4

- Scaling, irrigation, and cleaning

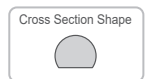

Maximum Power (est.)	
SC-3000	4
SC-7000	16

Code No.: 9186255

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.


Scaling

Scaling Tips

S11

Flat Tip S11

- Scaling, irrigation, and cleaning for wide areas (removing contaminants e.g. stains)

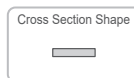

Maximum Power (est.)



SC-3000	6
SC-7000	20

Code No.: 8710600

S12

Flat Tip S12

- Scaling, irrigation, and cleaning for wide areas (removing contaminants e.g. stains)

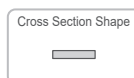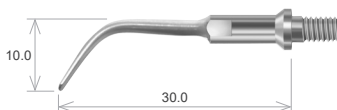

Maximum Power (est.)

SC-3000	6
SC-7000	20

Code No.: 8710627

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.





SP1


Solfy Point SP1

Connected to Endo Tip A EH1 or B EH2


- Calculus removal (mainly under the gum line)
- Fine adjustment of cavities
- Cleaning tooth surfaces

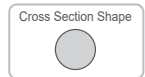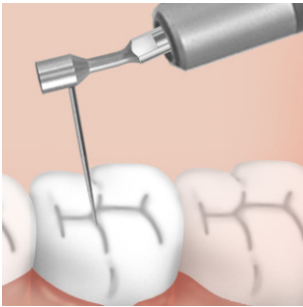

Solfy Point SP1
Code No.: 8713502 (Qty: 6)

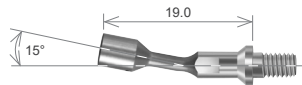

Endo Tip A EH1
Code No.: 8710805

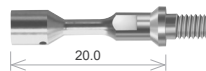

Endo Tip B EH2
Code No.: 8712204

Maximum Power (est.)	
SC-3000	6
SC-7000	15

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.


Perio

Periodontal Tips

P1

Curette Tip P1

- Scaling, irrigation, and cleaning for above and under the gum line and in root bifurcations.

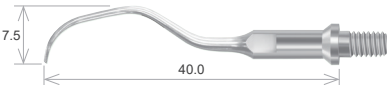



Maximum Power (est.)	
SC-3000	2
SC-7000	7

Code No.: 9186271

P2

Perio Tip P2

- Scaling, irrigation, and cleaning for above and under the gum line and in root bifurcations.

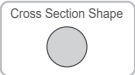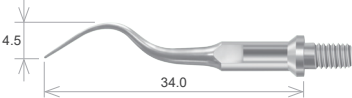

Maximum Power (est.)	
SC-3000	0.5
SC-7000	7

Code No.: 9186298



Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.



P3

Curette Tip P3

- Scaling, irrigation, and cleaning for above and under the gum line and in root bifurcations.

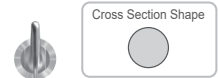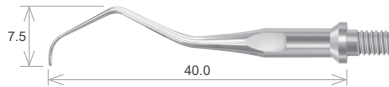



Maximum Power (est.)	
SC-3000	0.5
SC-7000	5

Code No.: 9186301

P4

Perio Tip P4

- Scaling, irrigation, and cleaning for above and under the gum line and in root bifurcations.

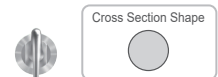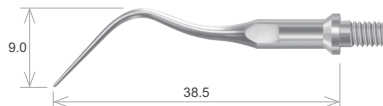

Maximum Power (est.)	
SC-3000	0.5
SC-7000	10

Code No.: 9186328

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.



Perio

Periodontal Tips

P5

Root Planing Tip P5

- Scaling, irrigation, and cleaning for above and under the gum line and in root bifurcations.

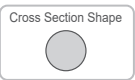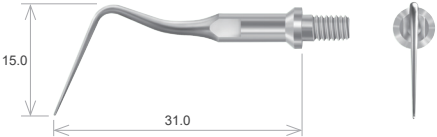

Maximum Power (est.)	
SC-3000	3
SC-7000	10

Code No.: 9186379



Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.






P20L / P20R

Furcation Tip P20L / P20R

- Scaling, irrigation, and cleaning for above and under the gum line and in root bifurcations.

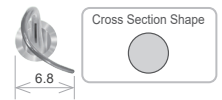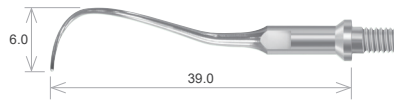

Furcation Tip P20L
Code No.: 9186344

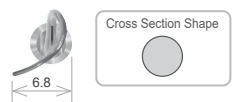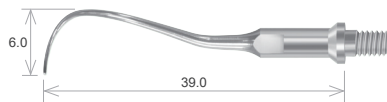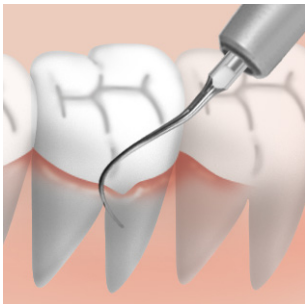

Furcation Tip P20R
Code No.: 9186336

Maximum Power (est.)	
SC-3000	0.5
SC-7000	7

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.




Perio

Periodontal Tips

P21L /P21R




Root Planing Tip P21L / P21R

- Scaling, irrigation, and cleaning for above and under the gum line and in root bifurcations.
- Tooth surface cleaning, root surface polishing, and root planing

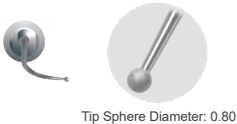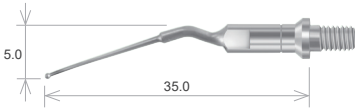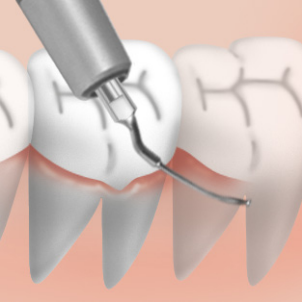

Tip Sphere Diameter: 0.80

Root Planing Tip P21L
Code No.: 8714509

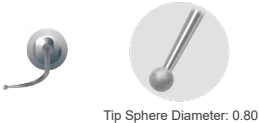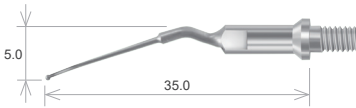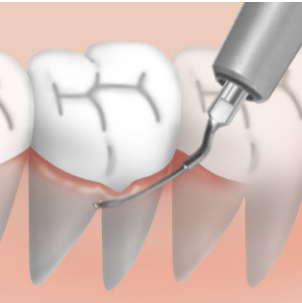

Tip Sphere Diameter: 0.80

Root Planing Tip P21R
Code No.: 8714487

Maximum Power (est.)	
SC-3000	5
SC-7000	10

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.



PP1

Round Point PP1


Connected to Endo Tip A EH1 or B EH2

- Calculus removal for root bifurcation area
- Irrigation for under the gum line
- Tooth surface cleaning, root surface polishing, and root planing



Tip Sphere Diameter: 0.85

Round Point PP1
Code No.: 8713901 (Qty: 3)

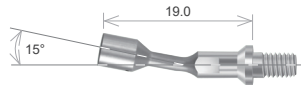

Endo Tip A EH1
Code No.: 8710805



Endo Tip B EH2
Code No.: 8712204

Maximum Power (est.)	
SC-3000	5
SC-7000	10

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.

Perio




Periodontal Tips

PP2L / PP2R


Perio Point PP2L / PP2R

Connected to Endo Tip M EH3


- Irrigation and calculus removal for under the gum line and root bifurcation area
- Tooth surface cleaning, root surface polishing, and root planing

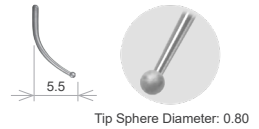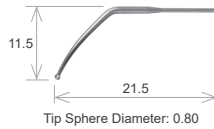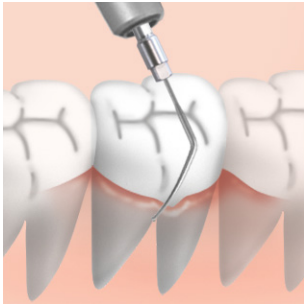

Perio Point PP2L
Code No.: 9186360 (Qty: 3)

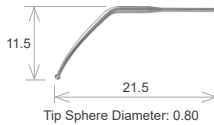

Perio Point PP2R
Code No.: 9186352 (Qty: 3)

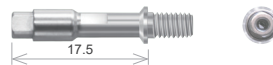

Endo Tip M EH3
Code No.: 8710848

Maximum Power (est.)	
SC-3000	3
SC-7000	7

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.




Dental Needle

Connected to Irrigation Connector P30

- Irrigation for periodontal pockets and removing plaque



Dental Needle*

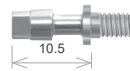

Irrigation Connector P30
Code No.: 8714002



Maximum Power (est.)	
SC-3000	6
SC-7000	10

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.

* Not manufactured by J. MORITA MFG. CORP. Contact your local dealer or J. MORITA OFFICE to purchase.




Endo

Endodontics Tips

E1

Root Canal Tip E1

- Remove cement and debris from canal
- Enlarge and irrigate canal
- Remove tooth substance (softened dentin, etc.)
- Enlarge and shape root canal orifice, fine adjustment of cavities

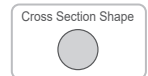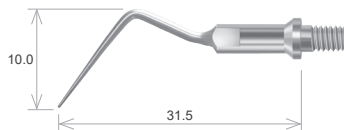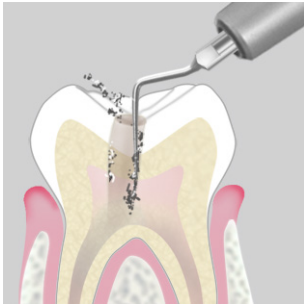




Maximum Power (est.)	
SC-3000	3
SC-7000	10

Code No.: 8713758

E2

Endo Tip E2

- Enlarge and irrigate canal
- Remove debris (cement, gutta-percha, sealer, fragments, etc.) from canals

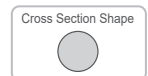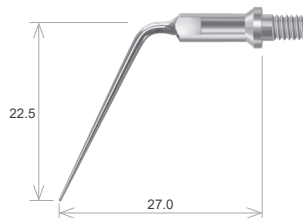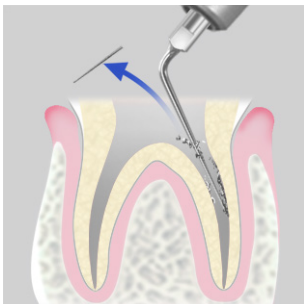

Maximum Power (est.)	
SC-3000	0.5
SC-7000	6

Code No.: 9186123



Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.



E3

Endo Tip E3

- Remove tooth substance (softened dentin, etc.)
- Enlarge and shape root canal orifice, fine adjustment of cavities

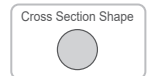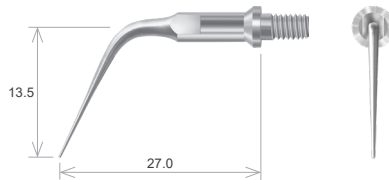




Maximum Power (est.)	
SC-3000	3
SC-7000	10

Code No.: 9186158

E4

Endo Tip E4

- Remove debris (cement, gutta-percha, sealer, fragments, etc.) from canals
- Remove tooth substance (softened dentin, etc.)
- Enlarge and shape root canal orifice, fine adjustment of cavities

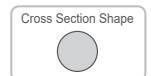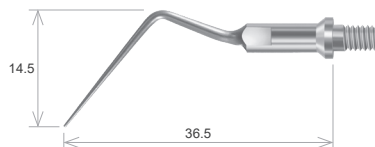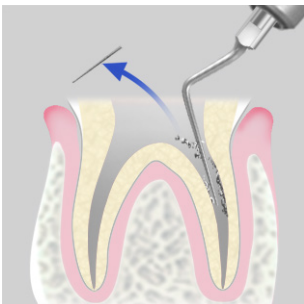

Maximum Power (est.)	
SC-3000	0.5
SC-7000	10

Code No.: 9186131

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.



Endo

Endodontics Tips

E10

Anchor Tip E10

- Remove debris (cement, gutta-percha, sealer, fragments, etc.) from canals
- Remove tooth substance (softened dentin, etc.)
- Enlarge and shape root canal orifice, fine adjustment of cavities

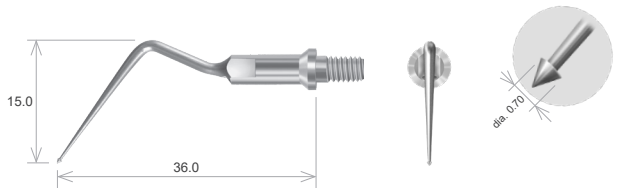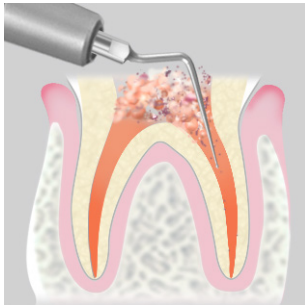



Code No.: 9186107

Maximum Power (est.)	
SC-3000	0.5
SC-7000	5

E20D

Round Tip E20D

- Enlarge fractured part of canal
- Remove debris (cement, gutta-percha, sealer, fragments, etc.) from canals
- Remove tooth substance (softened dentin, etc.)
- Enlarge and shape root canal orifice, fine adjustment of cavities

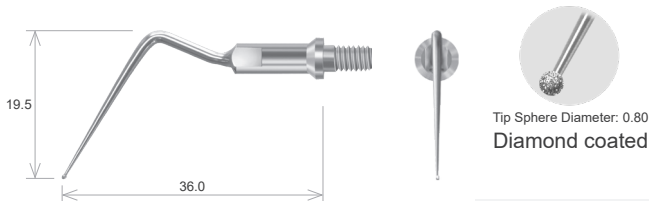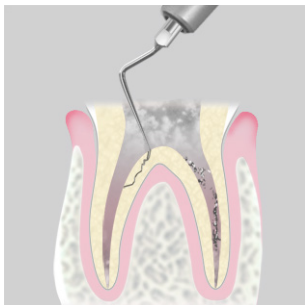

Code No.: 9186115

Maximum Power (est.)	
SC-3000	4
SC-7000	13

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.





EP1D

Diamond Point S EP1D


Connected to Endo Tip A EH1 or B EH2

- Enlarge canals
- Cut teeth, fine adjustment of cavities

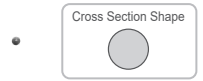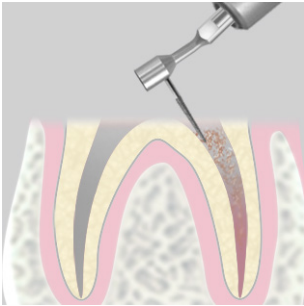


Diamond coated

Diamond Point S EP1D
Code No.: 8713448 (Qty: 3)

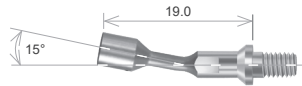

Endo Tip A EH1
Code No.: 8710805

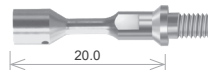

Endo Tip B EH2
Code No.: 8712204

Maximum Power (est.)	
SC-3000	5
SC-7000	10

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.

Endo




Endodontics Tips

EP2D

Diamond Point L EP2D


Connected to Endo Tip A EH1 or B EH2

- Enlarge canals
- Cut teeth, fine adjustment of cavities

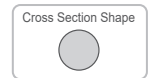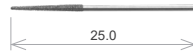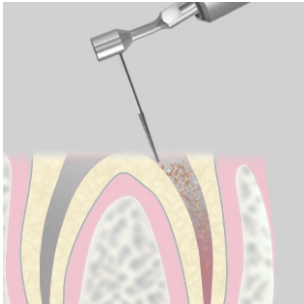

Diamond coated

Diamond Point L EP2D
Code No.: 8713405 (Qty: 3)

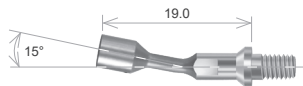

Endo Tip A EH1
Code No.: 8710805



Endo Tip B EH2
Code No.: 8712204

Maximum Power (est.)	
SC-3000	5
SC-7000	7


Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.







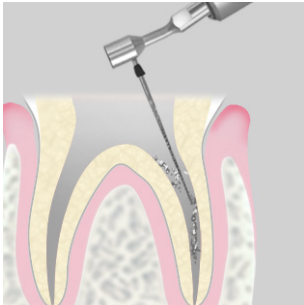

Solfy Ultrasonic File


Connected to Endo Tip A EH1 or B EH2

- Enlarge, irrigate, and clean canal
- Remove debris from canal




	File No.	Dimension
	EF15	27.7
	EF20	24.7
	EF25	26.2
	EF30	27.7
	EF35	25.7
	EF40	28.2

Solfy Ultrasonic File
Code No.: 8709807-ASS (Qty: 6 assorted files)

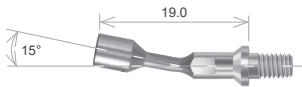

Endo Tip A EH1
Code No.: 8710805

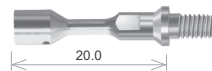

Endo Tip B EH2
Code No.: 8712204



File No.	Maximum Power (est.)	
	SC-3000	SC-7000T2
EF15	1.5	6
EF20	2	8
EF25	2.5	9
EF30	3	10
EF35	3	10
EF40	3	10

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.


Endo

Endodontics Tips


Ultrasonic Endo File (Single) Ultrasonic Diamond File (Single)

Connected to Endo Tip M EH3

- Remove tooth substance (softened dentin, etc.)
- Enlarge and shape root canal orifice, fine adjustment of cavities
- Remove debris (cement, gutta-percha, sealer, fragments, etc.) from canals

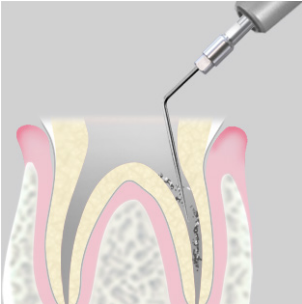


Ultrasonic Endo File (Single)*

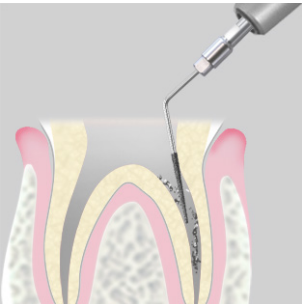

Diamond coated

Ultrasonic Diamond File (Single)*

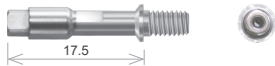

Endo Tip M EH3
Code No.: 8710848

Maximum Power (est.)	
SC-3000	3
SC-7000	10

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.


* Not manufactured by J. MORITA MFG. CORP. Contact your local dealer or J. MORITA OFFICE to purchase.




Ultrasonic Endo File (Double) Ultrasonic Diamond File (Double)

Connected to Endo Tip M EH3

- Remove tooth substance (softened dentin, etc.)
- Enlarge and shape root canal orifice, fine adjustment of cavities
- Remove debris (cement, gutta-percha, sealer, fragments, etc.) from canals

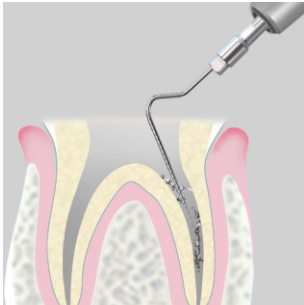

Ultrasonic Endo File (Double)*

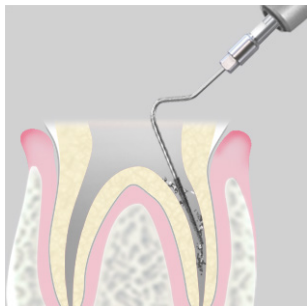

Diamond coated

Ultrasonic Diamond File (Double)*



Endo Tip M EH3
Code No.: 8710848

Maximum Power (est.)	
SC-3000	3
SC-7000	10

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.

* Not manufactured by J. MORITA MFG. CORP. Contact your local dealer or J. MORITA OFFICE to purchase.

Endo




Endodontics Tips


Dental Needle

Connected to Irrigation Connector P30

- Root canal irrigation

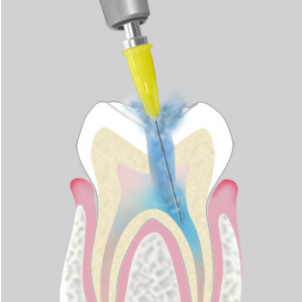

Dental Needle*

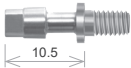

Irrigation Connector P30
Code No.: 8714002

Maximum Power (est.)	
SC-3000	6
SC-7000	10

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.

* Not manufactured by J. MORITA MFG. CORP. Contact your local dealer or J. MORITA OFFICE to purchase.






Retro Endo

Retro Tips (retrograde endodontics)

R1

Retro Tip R1

- Cut teeth
- Remove tooth substance
- Enlarge and shape root canal apex

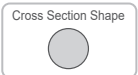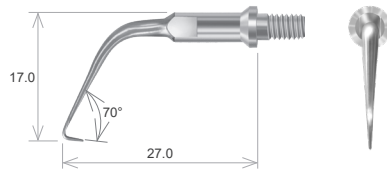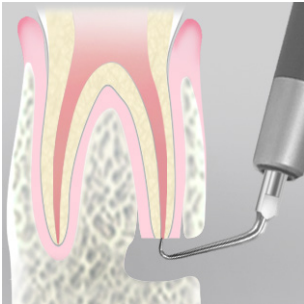




Maximum Power (est.)	
SC-3000	5
SC-7000	10

Code No.: 9186166

R2

Retro Tip R2

- Cut teeth
- Remove tooth substance
- Enlarge and shape root canal apex

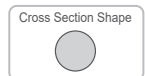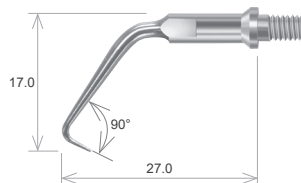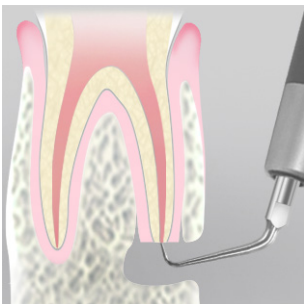

Maximum Power (est.)	
SC-3000	5
SC-7000	10

Code No.: 9186174

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.



Retro Endo

Retro Tips (retrograde endodontics)



R3L / R3R

Retro Tip R3L / R3R

- Cut teeth
- Remove tooth substance
- Enlarge and shape root canal apex

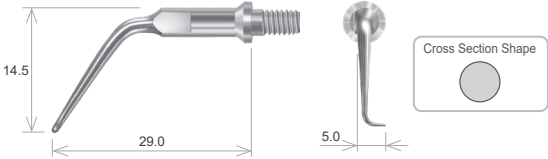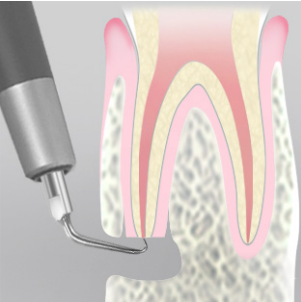

Retro Tip R3L
Code No.: 9186190

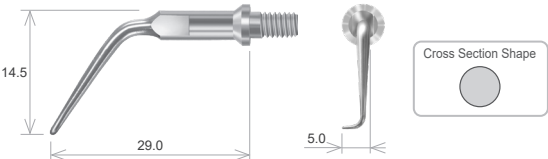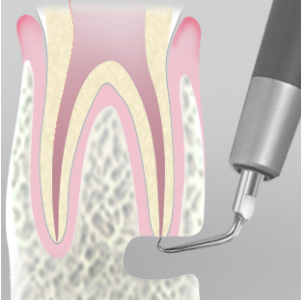

Retro Tip R3R
Code No.: 9186182

Maximum Power (est.)	
SC-3000	5
SC-7000	10

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.





RF1D


Retro File RF1D

Connected to Endo Tip M EH3


- Cut teeth
- Remove tooth substance
- Enlarge and shape root canal apex

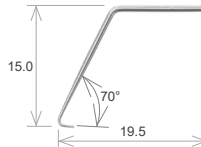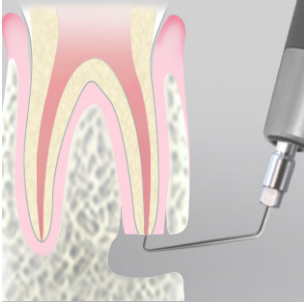

Retro File RF1D
Code No.: 9186204 (Qty: 3)

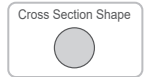

Diamond coated

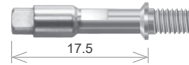

Endo Tip M EH3
Code No.: 8710848





Maximum Power (est.)	
SC-3000	0.5
SC-7000	7

RF2D


Retro File RF2D

Connected to Endo Tip M EH3

- Cut teeth
- Remove tooth substance
- Enlarge and shape root canal apex

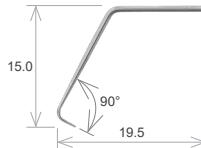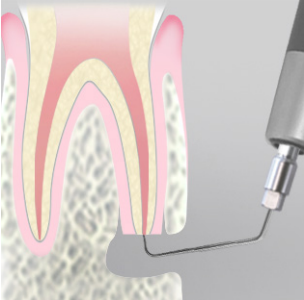

Retro File RF2D
Code No.: 9186212 (Qty: 3)

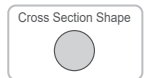

Diamond coated



Endo Tip M EH3
Code No.: 8710848



Maximum Power (est.)	
SC-3000	0.5
SC-7000	7

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.

Retro Endo






Retro Tips (retrograde endodontics)

RF3LD / RF3RD




Retro File RF3LD / RF3RD

Connected to Endo Tip M EH3

- Cut teeth
- Remove tooth substance
- Enlarge and shape root canal apex

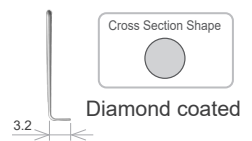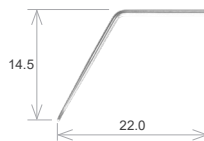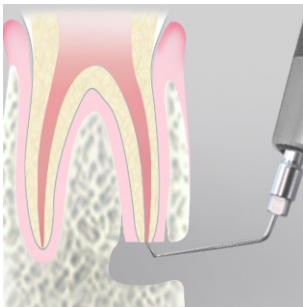

Retro File RF3LD
Code No.: 9186239 (Qty: 3)

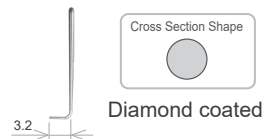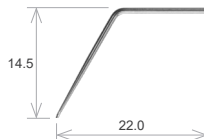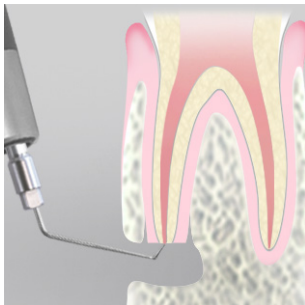

Retro File RF3RD
Code No.: 9186220 (Qty: 3)



Endo Tip M EH3
Code No.: 8710848

Maximum Power (est.)	
SC-3000	0.5
SC-7000	7

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.




Others

Other Tips

S20

Removal Tip S20

- Remove cast metal prosthetics, cement, amalgams, etc.

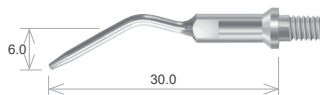

Cross Section Shape



Maximum Power (est.)



SC-3000	7
SC-7000	22

Code No.: 8710708

S30

Ultrasonic Scalpel Tip S30

- Incision and excision of periodontal tissue (alveolar bone and periodontal membrane)

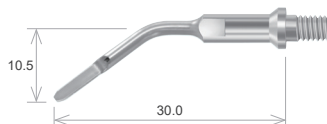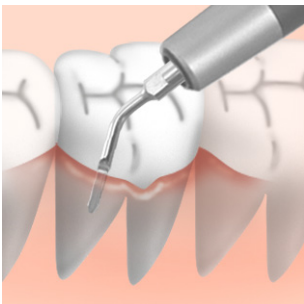

Cross Section Shape



Maximum Power (est.)

SC-3000	7
SC-7000	22

Code No.: 8714460

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.

Internal Thread Type

SC-7000T2





Scaling

Scaling Tips

TS1

Universal Tip TS1

- Scaling, irrigation, and cleaning

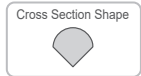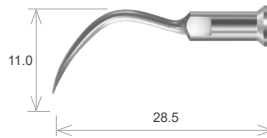

Code No.: 5252154



Maximum Power (est.)

SC-7000T2 | 20

TS2

Straight Tip TS2

- Calculus removal (mainly inside periodontal pocket)
- Irrigation (cavities, root canals, blind pockets, periodontal pockets)
- Debris removal (cement, gutta-percha, sealer, fragments, etc.) from canals

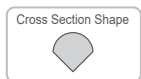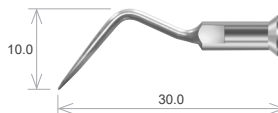

Code No.: 5252161

Maximum Power (est.)

SC-7000T2 | 20



Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.



TS3

Scaling Tip TS3

- Scaling, irrigation, and cleaning (mainly fissures)

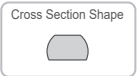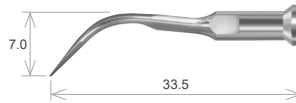

Maximum Power (est.)


SC-7000T2 | 20

Code No.: 5252155

TS4

Scaling Tip TS4

- Scaling, irrigation, and cleaning

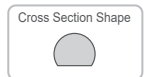

Maximum Power (est.)

SC-7000T2 | 16

Code No.: 5252156

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.

Scaling





Scaling Tips

SP1


Solffy Point SP1

Connected to Endo Tip A TEH1 or B TEH2


- Calculus removal (mainly under the gum line)
- Fine adjustment of cavities
- Cleaning tooth surfaces

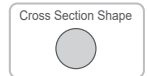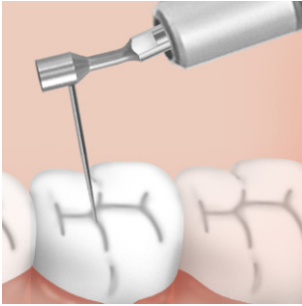

Solffy Point SP1
Code No.: 8713502 (Qty: 6)

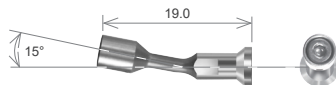

Endo Tip A TEH1
Code No.: 5252153

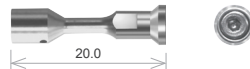

Endo Tip B TEH2
Code No.: 5252169

Maximum Power (est.)

SC-7000T2 | 15

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.




Perio

Periodontal Tips

TP1

Curette Tip TP1

- Scaling, irrigation, and cleaning for above and under the gum line and in root bifurcations.

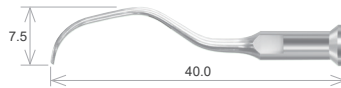

Maximum Power (est.)



SC-7000T2 | 7

Code No.: 5252159

TP2

Perio Tip TP2

- Scaling, irrigation, and cleaning for above and under the gum line and in root bifurcations.

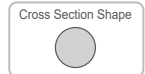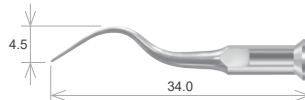

Maximum Power (est.)

SC-7000T2 | 7

Code No.: 5252162

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.

Perio

Periodontal Tips

TP3

Curette Tip TP3

- Scaling, irrigation, and cleaning for above and under the gum line and in root bifurcations.




Code No.: 5252163

Maximum Power (est.)	
SC-7000T2	5

TP4

Perio Tip TP4

- Scaling, irrigation, and cleaning for above and under the gum line and in root bifurcations.

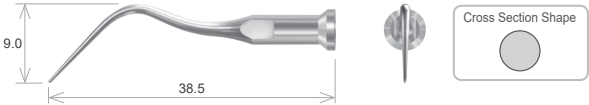

Code No.: 5252164

Maximum Power (est.)	
SC-7000T2	10


Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.



TP5

Root Planing Tip TP5

- Scaling, irrigation, and cleaning for above and under the gum line and in root bifurcations.

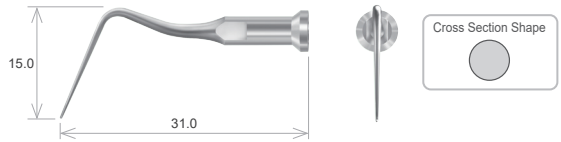

Code No.: 5252157

Maximum Power (est.)

SC-7000T2 | 10




Perio

Periodontal Tips




TP21L / TP21R

Root Planing Tip TP21L / TP21R

- Scaling, irrigation, and cleaning for above and under the gum line and in root bifurcations.
- Tooth surface cleaning, root surface polishing, and root planing

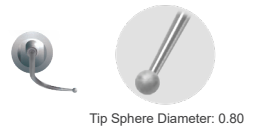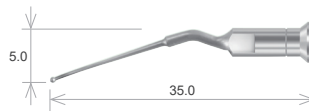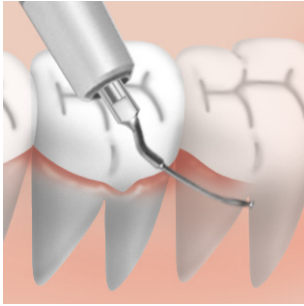

Root Planing Tip TP21L
Code No.: 5252167

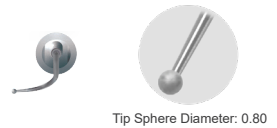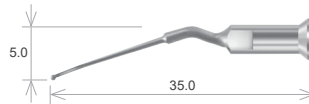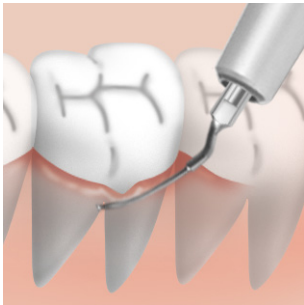

Root Planing Tip TP21R
Code No.: 5252165

Maximum Power (est.)

SC-7000T2 | 10

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.



PP1

Round Point PP1

Connected to Endo Tip A TEH1 or B TEH2


- Calculus removal for root bifurcation area
- Irrigation for under the gum line
- Tooth surface cleaning, root surface polishing, and root planing



Round Point PP1
Code No.: 8713901 (Qty: 3)



Tip Sphere Diameter: 0.85

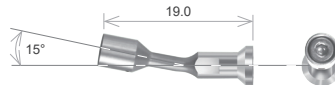

Endo Tip A TEH1
Code No.: 5252153



Endo Tip B TEH2
Code No.: 5252169

Maximum Power (est.)

SC-7000T2 | 10

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.

Perio






Periodontal Tips

PP2L / PP2R



Perio Point PP2L / PP2R

Connected to Endo Tip M TEH3

- Irrigation and calculus removal for under the gum line and root bifurcation area
- Tooth surface cleaning, root surface polishing, and root planing

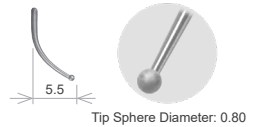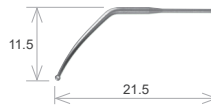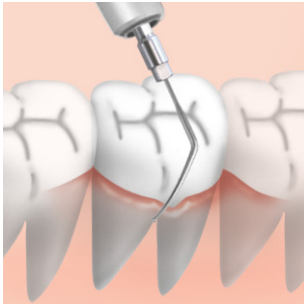

Perio Point PP2L
Code No.: 9186360 (Qty: 3)

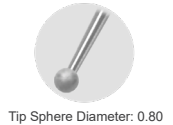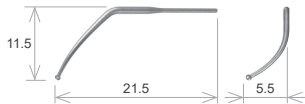

Perio Point PP2R
Code No.: 9186352 (Qty: 3)



Endo Tip M TEH3
Code No.: 5252160

Maximum Power (est.)
SC-7000T2 7

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.





Endo

Endodontics Tips

TE1

Root Canal Tip TE1

- Remove cement and debris from canal
- Enlarge and irrigate canal
- Remove tooth substance (softened dentin, etc.)
- Enlarge and shape root canal orifice, fine adjustment of cavities

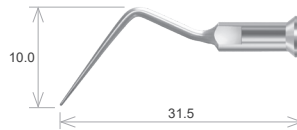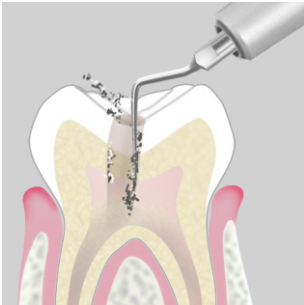




Root Canal Tip TE1
Code No.: 5252158

Maximum Power (est.)	
SC-7000T2	10

TE4

Endo Tip TE4

- Remove debris (cement, gutta-percha, sealer, fragments, etc.) from canals
- Remove tooth substance (softened dentin, etc.)
- Enlarge and shape root canal orifice, fine adjustment of cavities

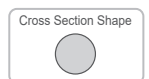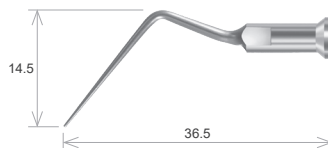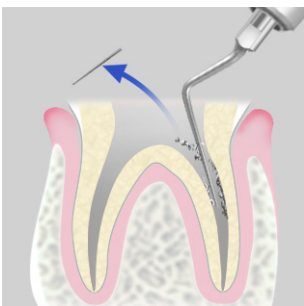

Endo Tip TE4
Code No.: 5252168

Maximum Power (est.)	
SC-7000T2	10

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.



Endo

Endodontics Tips

TE10

Anchor Tip TE10

- Remove debris (cement, gutta-percha, sealer, fragments, etc.) from canals
- Remove tooth substance (softened dentin, etc.)
- Enlarge and shape root canal orifice, fine adjustment of cavities

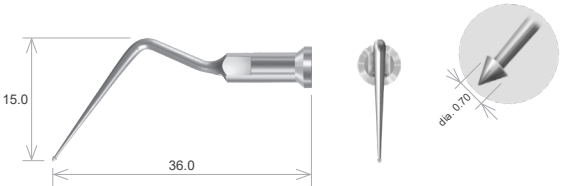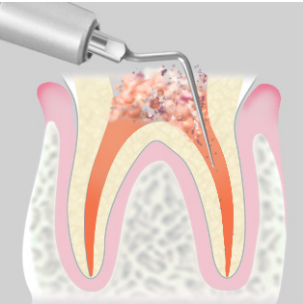

Anchor Tip TE10
Code No.: 5252166

Maximum Power (est.)	
SC-7000T2	5

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.





EP1D

Diamond Point S EP1D


Connected to Endo Tip A TEH1 or B TEH2

- Enlarge canals
- Cut teeth, fine adjustment of cavities

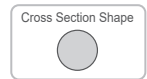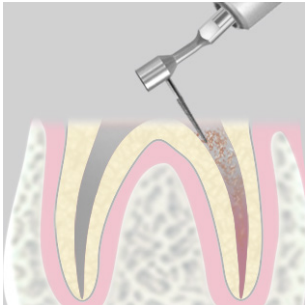


Diamond coated

Diamond Point S EP1D
Code No.: 8713448 (Qty: 3)

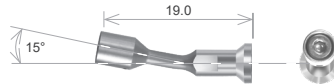

Endo Tip A TEH1
Code No.: 5252153

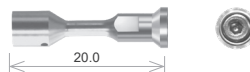

Endo Tip B TEH2
Code No.: 5252169

Maximum Power (est.)

SC-7000T2 | 10

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.

Endo



Endodontics Tips

EP2D


Diamond Point L EP2D

Connected to Endo Tip A TEH1 or B TEH2


- Enlarge canals
- Cut teeth, fine adjustment of cavities

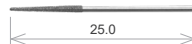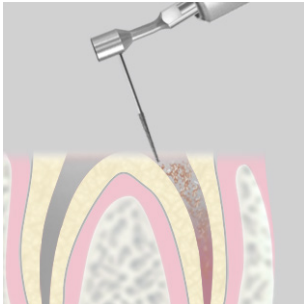


Diamond Point L EP2D
Code No.: 8713405 (Qty: 3)

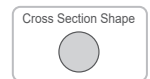

Diamond coated

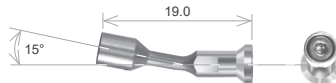

Endo Tip A TEH1
Code No.: 5252153

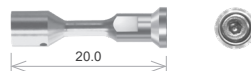

Endo Tip B TEH2
Code No.: 5252169

Maximum Power (est.)

SC-7000T2 | 7


Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.



Solfy Ultrasonic File


Connected to Endo Tip A TEH1 or B TEH2

- Enlarge, irrigate, and clean canal
- Remove debris from canal

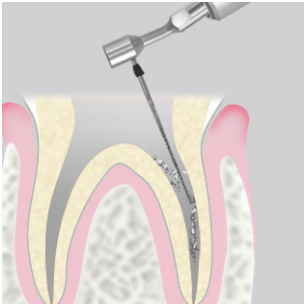


	File No.	Dimension
	EF15	27.7
	EF20	24.7
	EF25	26.2
	EF30	27.7
	EF35	25.7
	EF40	28.2

Solfy Ultrasonic File
Code No.: 8709807 (Qty: 6 assorted files)

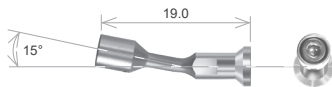

Endo Tip A TEH1
Code No.: 5252153

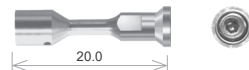

Endo Tip B TEH2
Code No.: 5252169

Maximum Power (est.)	
File No.	SC-7000T2
EF15	6
EF20	8
EF25	9
EF30	10
EF35	10
EF40	10

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.




Endo

Endodontics Tips

Ultrasonic Endo File (Single) Ultrasonic Diamond File (Single)

Connected to Endo Tip M TEH3

- Remove tooth substance (softened dentin, etc.)
- Enlarge and shape root canal orifice, fine adjustment of cavities
- Remove debris (cement, gutta-percha, sealer, fragments, etc.) from canals

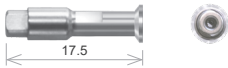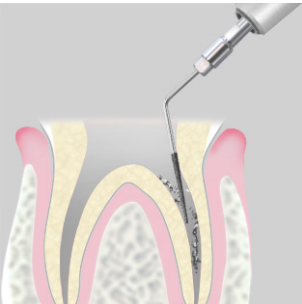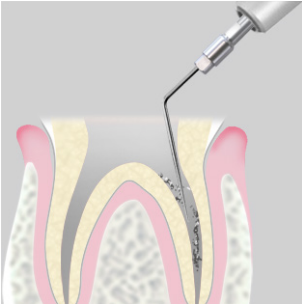

Endo Tip M TEH3
Code No.: 5252160

Maximum Power (est.)	
SC-7000T2	10

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.
* Not manufactured by J. MORITA MFG. CORP. Contact your local dealer or J. MORITA OFFICE to purchase.




Ultrasonic Endo File (Double)


Ultrasonic Diamond File (Double)

Connected to Endo Tip M TEH3

- Remove tooth substance (softened dentin, etc.)
- Enlarge and shape root canal orifice, fine adjustment of cavities
- Remove debris (cement, gutta-percha, sealer, fragments, etc.) from canals

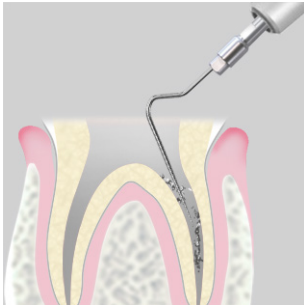


Ultrasonic Endo File (Double)*

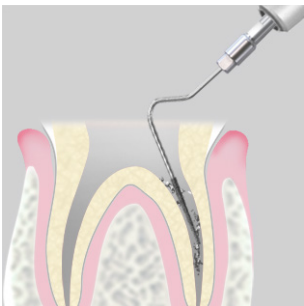

Diamond coated

Ultrasonic Diamond File (Double)*

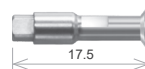

Endo Tip M TEH3
Code No.: 5252160



Maximum Power (est.)	
SC-7000T2	10

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.

Retro Endo




Retro Tips (retrograde endodontics)

RF1D

Retro File RF1D

Connected to Endo Tip M TEH3

- Cut teeth
- Remove tooth substance
- Enlarge and shape root canal apex

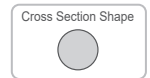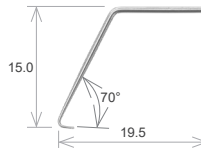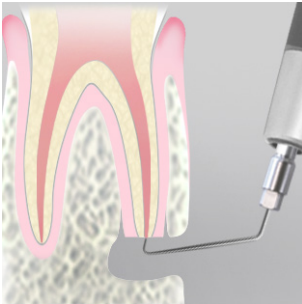

Diamond coated

Retro File RF1D
Code No.: 9186204 (Qty: 3)



Endo Tip M TEH3
Code No.: 5252160

Maximum Power (est.)




SC-7000T2 | 7

RF2D

Retro File RF2D

Connected to Endo Tip M TEH3

- Cut teeth
- Remove tooth substance
- Enlarge and shape root canal apex

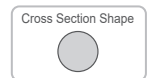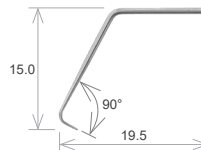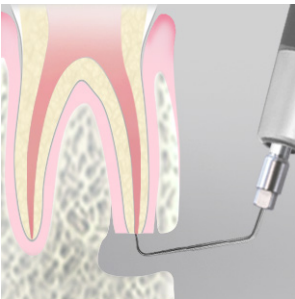

Diamond coated

Retro File RF2D
Code No.: 9186212 (Qty: 3)



Endo Tip M TEH3
Code No.: 5252160

Maximum Power (est.)

SC-7000T2 | 7

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.





RF3LD / RF3RD



Retro File RF3LD / RF3RD

Connected to Endo Tip M TEH3

- Cut teeth
- Remove tooth substance
- Enlarge and shape root canal apex

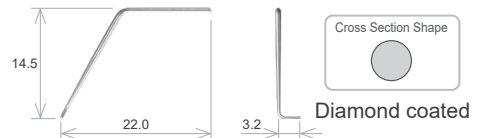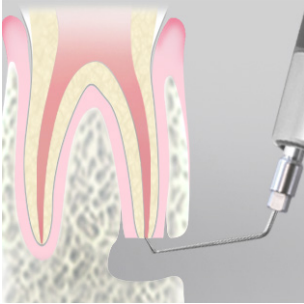

Retro File RF3LD
Code No.: 9186239 (Qty: 3)

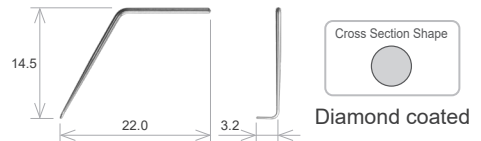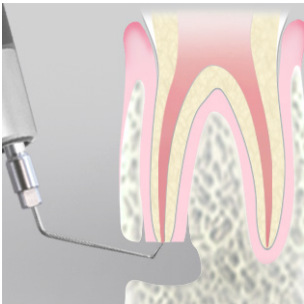

Retro File RF3RD
Code No.: 9186220 (Qty: 3)



Endo Tip M TEH3
Code No.: 5252160



Maximum Power (est.)

SC-7000T2 | 7

Use the accompanying torque wrench or tip tool to attach and remove a scaler tip.

Others






Other Tips

TS20

Removal Tip TS20

- Remove cast metal prosthetics, cement, amalgams, etc.

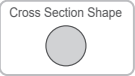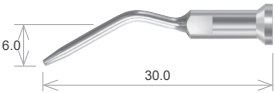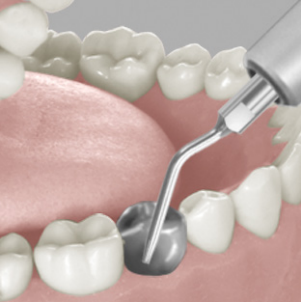

Removal Tip TS20
Code No.: 5252170

Maximum Power (est.)	
SC-7000T2	22

Dimensions (unit: mm) are standard ones; there may be some slight discrepancies for individual tips due to dimensional tolerances.

Consumable and Replacement Parts

Tip Replacement Parts

All tip replacement parts are commonly used for both thread types.

Torque Wrench



Code No.: 8711887

Tip Tool




Code No.: 8711402

File Tool



Code No.: 7500170

Tip Wear Gauge

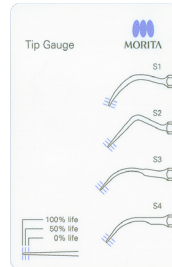

Code No.: 8712190

Tip Power Guide



Do not use scaler tips in excess of their maximum (est.) power level. Using the scaler at powers higher than those recommended or applying the scaler tip with excessive force could result in overheating the treatment area or scaler tip, and damaging the scaler tip.

Scaler Tip Name		Identification Number	Maximum Power (est.)		
			SC-3000	SC-7000	SC-7000T2
Scaling	Universal Tip	S1	6	20	—
		TS1	—	—	20
	Straight Tip	S2	6	20	—
		TS2	—	—	20
	Scaling Tip	S3	6	20	—
		TS3	—	—	20
	Scaling Tip	S4	4	16	—
		TS4	—	—	16
Flat Tip	S11	6	20	—	
Flat Tip	S12	6	20	—	
Solfy Point *1	SP1	6	15	15	
Perio	Curette Tip	P1	2	7	—
		TP1	—	—	7
	Perio Tip	P2	0.5	7	—
		TP2	—	—	7
	Curette Tip	P3	0.5	5	—
		TP3	—	—	5
	Perio Tip	P4	0.5	10	—
		TP4	—	—	10
	Root Planing Tip	P5	3	10	—
		TP5	—	—	10
	Furcation Tip	P20L/P20R	0.5	7	—
	Root Planing Tip	P21L/P20R	5	10	—
		TP21L/TP21R	—	—	10
	Dental Needle *2, *4	—	6	10	—
Round Point *1	PP1	5	10	10	
Perio Point *3	PP2L/PP2R	3	7	7	
Endo	Root Canal Tip	E1	3	10	—
		TE1	—	—	10
	Endo Tip	E2	0.5	6	—
	Endo Tip	E3	3	10	—
		E4	0.5	10	—
	Anchor Tip	E10	0.5	5	—
	TE10	—	—	5	


Scaler Tip Name		Identification Number	Maximum Power (est.)		
			SC-3000	SC-7000	SC-7000T2
Endo	Round Tip	E20D	4	13	—
	Diamond Point S *1	EP1D	5	10	10
	Diamond Point L *1	EP2D	5	7	7
	Solfy File #15 *1	EF15	1.5	6	6
	Solfy File #20 *1	EF20	2	8	8
	Solfy File #25 *1	EF25	2.5	9	9
	Solfy File #30 *1	EF30	3	10	10
	Solfy File #35 *1	EF35	3	10	10
	Solfy File #40 *1	EF40	3	10	10
	Ultrasonic Endo File (Single) *3, *4	—	3	10	10
	Ultrasonic Diamond File (Single) *3, *4	—	3	10	10
	Ultrasonic Endo File (Double) *3, *4	—	3	10	10
	Ultrasonic Diamond File (Double) *3, *4	—	3	10	10
	Dental Needle *2, *4	—	6	10	—
	Retro Tip	R1	5	10	—
	Retro Tip	R2	5	10	—
	Retro Tip	R3L/R3R	5	10	—
	Retro File *3	RF1D	0.5	7	7
	Retro File *3	RF2D	0.5	7	7
	Retro File *3	RF3LD/RF3RD	0.5	7	7
	Endo Tip A	EH1	—	—	—
TEH1		—	—	—	
Endo Tip B	EH2	—	—	—	
	TEH2	—	—	—	
Endo Tip M	EH3	—	—	—	
	TEH3	—	—	—	
Others	Removal Tip	S20	7	22	—
		TS20	—	—	22
	Ultrasonic Scalpel Tip	S30	7	22	—

*1 Use connected to Endo Tip A or B.

*2 Use connected to Irrigation Connector P30.

*3 Use connected to Endo Tip M.

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*4 Not manufactured by J. MORITA MFG. CORP. Contact your local dealer or J. MORITA OFFICE to purchase.





Development and Manufacturing

J. MORITA MFG. CORP.

680 Higashihama Minami-cho, Fushimi-ku,
Kyoto 612-8533, Japan
T +81. (0)75. 611 2141, F +81. (0)75. 622 4595

Morita Global Website
www.morita.com

Distribution

J. MORITA CORP.

3-33-18 Tarumi-cho, Suita-shi, Osaka 564-8650, Japan
T +81. (0)6. 6380 1521, F +81. (0)6. 6380 0585

J. MORITA USA, INC.

9 Mason, Irvine CA 92618, USA
T +1. 949. 581 9600, F +1. 949. 581 8811

J. MORITA EUROPE GMBH

Justus-von-Liebig-Strasse 27b, 63128 Dietzenbach, Germany
T +49. (0)6074. 836 0, F +49. (0)6074. 836 299

MORITA DENTAL ASIA PTE. LTD.

150 Kampong Ampat #06-01A KA Centre, Singapore 368324
T +65. 6779. 4795, F +65. 6777. 2279

J. MORITA CORP. AUSTRALIA & NEW ZEALAND

Suite 2.05, 247 Coward Street, Mascot NSW 2020, Australia
T +61. (0)2. 9667 3555, F +61. (0)2. 9667 3577

J. MORITA CORP. MIDDLE EAST

4 Tag Al Roasaa, Apartment 902, Saba Pacha 21311 Alexandria, Egypt
T +20. (0)3. 58 222 94, F +20. (0)3. 58 222 96

J. MORITA CORP. INDIA

Filix Office No.908, L.B.S. Marg, Opp. Asian Paints, Bhandup (West), Mumbai 400078, India
T +91-82-8666-7482

J. MORITA MFG. CORP. INDONESIA

28F, DBS Bank Tower, Jl. Prof. Dr. Satrio Kav. 3-5, Jakarta 12940, Indonesia
T +62-21-2988-8332, F + 62-21-2988-8201

SIAMDENT CO., LTD.

71/10 Moo 5 T, Tharkham A. Bangpakong Chachuengsao 24130 Thailand
T +66 (0) 3857 3042, F +66 (0) 3857 3043

www.siamdent.com

EU Authorized Representative under the European Directive 93/42/EEC

EC REP Medical Technology Promedt Consulting GmbH

Ernst-Heckel-Straße 7, 66386 St. Ingbert, Germany T +49. 6894 581020, F +49. 6894 581021

The authority granted to the authorized representative, Medical Technology Promedt Consulting GmbH, by J. MORITA MFG. CORP. is solely limited to the work of the authorized representative with the requirements of the European Directive 93/42/EEC for product registration and incident report.

Diagnostic and Imaging Equipment

Treatment Units

Handpieces and Instruments

Endodontic Systems

Laser Equipment

Laboratory Devices

Educational and Training Systems

Auxiliaries