



Veraview iX

Intraoral Imaging

- Accurate LED Positioning Guide
- High Frequency DC Generator reduces Patient Dosage
- Optimized Time Settings for Film and Digital Exposures
- Simplified Operation with Illuminated Pictogram Menu
- Easy Seat Height Adjustment



Thinking ahead. Focused on life.

LED Positioning Guide Eliminates Guesswork for Accurate Alignment




iX-R version with patient's chair

- LED Positioning Guide**
 - Two LED light beams converge into a single point for visual confirmation of correct alignment for accurate X-rays.
 - The cone is short and easy to manipulate which also facilitates correct positioning. It is also transparent for superior visibility.
 - The lightweight head and arm can easily be maneuvered with a delicate touch.

Order no.

1242 Veraview iX Type R

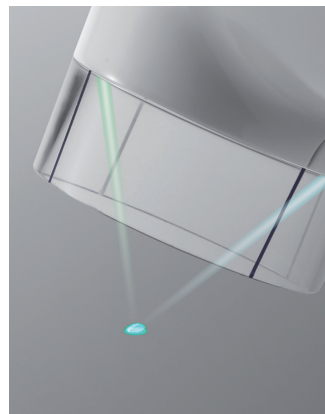

- Constant Voltage DC Generator**
 - By utilizing a high frequency, direct current (DC) generator, this unit produces 'hard' X-rays which offer reduced dosage compared to units that operate with an alternating current.
 - Tube voltage can be set at 60 kV or 70 kV.

Exposure Settings


Digital or film scan options are available. An optimized exposure time is set after parameter selections are made.

Illuminated Pictogram Menu

Menu items illuminate to confirm selections and help prevent mistakes.

Easy Seat Height Adjustment (Type R)

Seat height is easily adjusted by pulling up the lever for the gas cylinder under the seat.

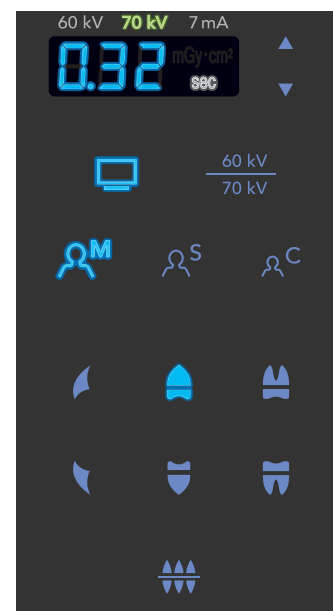

iX-WA wall-mounted version

Order no.

1220 Veraview iX Type WA550

1230 Veraview iX Type WA800

1235 Veraview iX Type WA1000

Specifications

Model	Veraview iX (wall-mounted version)	Veraview iX-R (version with patient's chair)
Voltage	100-120 / 220-240 VAC	100-120 / 220-240 VAC
Power Consumption	1,9 kVA	1,9 kVA
Tube Potential	60 / 70 kV	60 / 70 kV
Tube Current 7 mA	7 mA	7 mA
Exposure Time	0.01~2 seconds	0.01~2 seconds
Weight	approx. 23 kg	approx. 25 kg

Development and Manufacturing

J. MORITA MFG. CORP

680 Higashihama Minami-cho, Fushimi-ku,
Kyoto 612-8533, Japan
T +81. (0)75. 611 2141, F +81. (0)75. 622 4595

Morita Global Website
www.morita.com

Distribution

J. MORITA CORP

3-33-18 Tarumi-cho, Suita-shi, Osaka 564-8650, Japan
T +81. (0)6. 6380 1521, F +81. (0)6. 6380 0585

J. MORITA USA, INC.

9 Mason, Irvine CA 92618, USA
T +1. 949. 581 9600, F +1. 949. 581 8811

Subject to technical changes and errors.
JME EN FA0921 *0.0 Pub: No. DI165-B00004-EN-1

J. MORITA EUROPE GMBH

Justus-von-Liebig-Strasse 27b, 63128 Dietzenbach, Germany
T +49. (0)6074. 836 0, F +49. (0)6074. 836 299

MORITA DENTAL ASIA PTE. LTD.

3 Science Park Drive,
#01-05 The Franklin Singapore Science Park1, Singapore 118223
T +65. 6779. 4795, F +65. 6777. 2279

J. MORITA CORPORATION AUSTRALIA & NEW ZEALAND

Suite 2.05, 247 Coward Street, Mascot NSW 2020, Australia
T +61. (0)2. 9667 3555, F +61. (0)2. 9667 3577

J. MORITA CORP. MIDDLE EAST

4 Tag Al Roasaa, Apartment 902, Saba Pacha 21311 Alexandria, Egypt
T +20. (0)3. 58 222 94, F +20. (0)3. 58 222 96

SIAMDENT CO., LTD

71/10 Mu 5, Thakham, Bangpakong, Chachuengsao 24130, Thailand
T +66. (0)38. 573042, F +66. (0)38. 573043, www.siamdent.com